

## User Manual - BabyAir

### Operation

To operate BabyAir™:

1. Insert the Nebulizer Assembly into the hood through the hood top.
2. Remove the UDN cover (5) from the UDN body (7) by twisting counter-clockwise.
3. Fill the space in the UDN body (7) with the amount of liquid medication, measured exactly as prescribed by the physician (see picture).
4. Screw the UDN cover (5) and UDN body (7) back together until tightly sealed.
5. Attach the air hose (4) between the compressor (not shown) and the UDN cover (5).
6. Place the hood over the baby so that the head and chest are covered - the opening flap allows the rest of the body to lie comfortably outside the hood.
7. Set the funnel tip (9) close to the nose and mouth of the child- recommended distance is about 2 cm (one inch).
8. Start the compressor, and continue to run it until the medicine has been totally delivered (2 cc of medicine will take about 10 minutes).
9. On completion of treatment, typically 10-15 minutes, turn off the compressor; remove the hood from over the baby.
10. Prepare the system for its next use (see Cleaning and Assembly instructions).



**Figure 13.1** Parts and components of the BabyAir™



### **Cleaning and Assembly**

The following actions are required before using BabyAir™ for the first time, and between each use:

#### **The Nebulizer**

- Separate the Nebulizer Assembly from the hood.
- Remove the UDN cover (5) by twisting counter-clockwise.
- Extract the UDN funnel (6) by pulling it from the cover (5).
- Extract the UDN body (7) by pulling it from the funnel pipe (8).
- Extract the funnel tip (9) by pulling it from the funnel pipe (8).

**After disassembly** clean all parts with dishwashing soap (Dawn) and hot water, using a soft sponge only.

- Rinse well with clean water and allow drying prior to storage.

**When all the parts are dry**, assemble the nebulizer in reverse order, as follows:

- Connect the funnel tip (9) to the funnel pipe (8).
- Insert the UDN body (7) into the funnel pipe (8). Make sure that the two small pegs at the base of the UDN body fit into the slots at the head of the funnel pipe.
- Insert the UDN funnel (6) into the UDN body (7), with the open side facing down.



- Screw the UDN cover (5) onto the body (7) in a clockwise direction. Make sure the UDN cover (5) and UDN body (7) are tightly sealed.

The Nebulizer Assembly is now ready for operation.

### **The Hood**

- Hold the top of the hood and raise a leg (1) into its locking position (a click will be heard). Repeat this action for all four legs.
- Clean the plastic shield (2) with a damp cloth only.
- For convenient storage of the hood after use, hold the top of the hood and fold the legs inward.

### **Parts (Refer to Figure 13.1)**

#### **BabyAir™ operates with three main components:**


1. Transparent Hood - Covering shield draped over a plastic base, designed to let the baby inhale medication in total comfort.
2. Nebulizer Assembly – The Nebulizer uniquely designed converts the liquid medication into a fine aerosol mist and directs the aerosol toward the baby efficiently and safely.
3. Compressor - Not included as part of the package, and must be purchased separately.

#### **The Hood**

- 1. Legs**
- 2. Covering Shield**
- 3. Opening flap**

#### **The Nebulizer**

- 4. Air hose**
- 5. UDN Cover**
- 6. UDN Funnel**
- 7. UDN Body**
- 8. Funnel Pipe**
- 9. Funnel Tip**

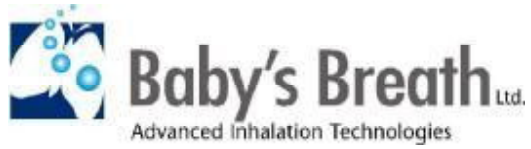

We at Baby's Breath thank you for purchasing BabyAir™, the unique inhalation system for newborns and infants.

There is no direct contact between the system and your child, which eliminates the discomfort and trapped feeling caused by the use of a regular facemask nebulizer combination.



**Important!**

**Read the User Guide completely before operating BabyAir™**

**Caution:**

1. Rx. Federal (USA) law restricts this device to sale by or on order of a physician.
2. Adult supervision during the entire treatment.
3. Remove the Hood once treatment is complete.
4. BabyAir™ includes small parts. When the device is disassembled, keep all parts out of the reach of children.
5. Do not remove any parts supplied with the system, and do not use any added or replacement parts that were not supplied by Baby's Breath Ltd.
6. Do not use BabyAir™ if damaged, or if any part does not appear to be working properly.

**Recommended Flow Rate:** 6 - 11 liters per minute.

BabyAir™ is intended for single patient, multi- use. To avoid spreading infection, do not transfer the system from one patient to another.

To answer any questions about the product or its operation, please contact Baby's Breath Ltd.

Telephone: +972-4-6262551

E-mail: [info@bbreath.com](mailto:info@bbreath.com)

Further product information is available on our website: [www.bbreath.com](http://www.bbreath.com).

**CE0483**